## Reagenti supplementari per A.F. GENITAL SYSTEM

### DESCRIZIONE

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F00012  
Rev.1 / 25.08.2009

## Additional reagents for A.F. GENITAL SYSTEM

### DESCRIPTION

**A.F. GENITAL SYSTEM - REAGENT** is a kit containing the reagents **KOVACS REAGENT**, **OXIDASE TEST**, **VASELINE OIL** for the performance respectively of Indole test, Oxidase test and for the preservation of a sufficiently anaerobic environment for the development of microorganisms to be detected by the test **A.F. GENITAL SYSTEM** (code 74156).

### CONTENT OF THE PACKAGES

Each package contains:

1 bottle of <b>KOVACS REAGENT</b>	2 bottles of <b>VASELINE OIL</b>
100 sticks of <b>OXIDASE TEST</b>	1 dropping bottle
1 instruction sheets	

### ITEMS NECESSARY BUT NOT INCLUDED IN THE PACKAGES

- A.F. GENITAL SYSTEM (code 74156)
- PHYSIOLOGICAL SOLUTION (code 20095)
- Sundry microbiology laboratory materials

### PRINCIPLE OF THE METHOD

**KOVACS REAGENT** is the reagent used for the performance of Indole test to confirm the presence of *Escherichia coli* in the **A.F. GENITAL SYSTEM** (code 74156). **OXIDASE TEST** is used for the rapid test for the cytochrome-oxidase enzyme detection. The test is performed to confirm the presence of *Neisseria gonorrhoeae* in the well **22-NES** in the test **A.F. GENITAL SYSTEM** (code 74156).

**VASELINE OIL** is used for the preservation of a sufficiently anaerobic environment in the wells **1-GR+**, **2-GR++**, **3-GR+++**, **4-ADC**, **5-UR**, **7-TE**, **8-PEF**, **9-OFX**, **10-DO**, **11-E**, **12-CLA**, **13-MN**, **14-JOS**, **15-CD**, **24-CAN** in the test **A.F. GENITAL SYSTEM** (code 74156).

### COMPOSITION

<b>VASELINE OIL:</b> Vaseline oil for microbiological use
<b>OXIDASE TEST:</b> Sticks impregnated with the reagent tetramethyl-p-phenylenediamine chlorhydrate
<b>KOVACS REAGENT</b> (content per bottle): P-dimetilaminobenzaldehyde 0,5 g HCl 2,5 mL Ethyl alcol 7,5 mL

### TEST PROCEDURE AND INTERPRETATION OF RESULTS

For the test procedure and the interpretation of results refer to instruction sheet of the test **A.F. GENITAL SYSTEM** (code 74156).

### QUALITY CONTROL FOR THE USER

Appearance: clear, limpid solutions.

The sticks are made of plastic with white colour absorbent paper at one extremity.

Microbiological control:

Every batch of **A.F. GENITAL SYSTEM - REAGENT** is subjected to the microbiological control, using the following microorganisms:

KOVACS REAGENT			
INDOLE TEST			
Microorganism	Colour	Result	
<i>Escherichia coli</i> ATCC 25922	Purple -red	positive test	
<i>Proteus mirabilis</i> ATCC 25933	Colorless	negative test	

OXIDASE TEST		
OXIDASE TEST		
Microorganism	Colour	Result
<i>Pseudomonas aeruginosa</i> ATCC 27853	blue	positive test
<i>Escherichia coli</i> ATCC 25922	colourless	negative test

VASELINE OIL		
ANAEROBIC ENVIRONMENT		
Microorganism	Result	
<i>Mycoplasma hominis</i> ATCC 23114	growth	positive test

### PRECAUTIONS

The product **A.F. GENITAL SYSTEM - REAGENT** is classifiable as hazardous under current legislation; it is recommended that the Safety Data Sheet be consulted on its use. **A.F. GENITAL SYSTEM - REAGENT** must be used only for *in vitro* diagnostic use. It is intended for use in a professional environment and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

### STORAGE

Store at 2-8 °C in its original package. In such conditions the product is valid until the expiry date shown on the label. Do not use them beyond that date. Dispose of them if they show signs of deterioration.

### DISPOSAL OF USED MATERIAL

After the use, **A.F. GENITAL SYSTEM - REAGENT** and the material that has come into contact with the sample must be decontaminated and disposed of in accordance with the laboratory procedures for the decontamination and disposal of potentially infected material.

### REFERENCES

1. Murray, Baron, Pfaller, Tenorev and Tenover: *Manual of Clinical Microbiology* (1995).
2. Bayley and Scott's: *Diagnostic Microbiology* (1986).
3. Edwin H. Lennette: *Manual of Clinical Microbiology* (1995).

### PRESENTATION

Product	REF	$\Sigma$
<b>A.F. GENITAL SYSTEM - REAGENT</b>	80258	100 tests

### TABLE OF SYMBOLS

<b>IVD</b> In Vitro Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
<b>REF</b> Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	<b>LOT</b> Batch code

**LIOFILCHEM Bacteriology Products**

Via Scozia Zona Ind.le - 64026 Roseto D.A. (TE) - Italy

Tel. +390858930745

Fax +390858930330

Website: [www.liofilchem.net](http://www.liofilchem.net)

E-Mail: [liofilchem@liofilchem.net](mailto:liofilchem@liofilchem.net)

F00012  
Rev.1 / 25.08.2009