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E-Mail: liofilchem@liofilchem.net

F12211
Rev.0 / 17.01.2005

Biological indicators of steam sterilization processes with *Geobacillus stearothermophilus* (ATCC 7953) spores contained in ampoules

DESCRIPTION

USP (United States Pharmacopoeia), EP (European Pharmacopoeia) and DAB (Deutsches Arzneibuch) standards recommend to use bioindicators during steam sterilization processes of pharmaceuticals, drugs in vials, culture media, medical instruments and more. Biological indicators **STERIL CONTROL GST E6/E5/E4 ampoules** are used for regular control of steam sterilization cycles (for ie. 15 minutes at 121 °C) and control of any steam autoclave functionality. Biological indicators **STERIL CONTROL GST E6/E5/E4 ampoules** are produced under strictly controlled conditions in order to satisfy the requirements in the USP current edition and in accordance with ISO 11138 and EN 866 standards.

PRINCIPLE

STERIL CONTROL GST E6/E5/E4 ampoules contain 4 ml of a nutritive broth, designed to this purpose, made of casein and soy seeds enzymatic digests and *Geobacillus stearothermophilus* (ATCC 7953) spores in predefined concentrations: E6=1-5x10⁶CFU/ampoule; E5=1-5x10⁵CFU/ampoule; E4=1-5x10⁴CFU/ampoule.

Spores in the ampoules are completely killed off after 15 minutes of sterilization at 121°C and therefore during the following incubation they are not able to grow and to modify the medium's colour. At a lower temperature or at a shorter sterilization time, spores partially survive and therefore during the incubation are able to grow and to modify the medium's colour.

INSTRUCTIONS FOR USE

- Take one or more ampoules from the package.
- Put the ampoules on the bottom, in the centre, inner sides and on the critical points of the autoclave.
- For autoclaves with capacity up to 250 litres put two ampoules for each selected point of the autoclave. For autoclaves with capacity higher than 250 litres put six or more ampoules in each selected point.
- Remove the ampoules after sterilization and incubate at 55-60 °C (131-140 °F) for 24-48 hours or for a different time validated by user.
- Incubate, at the same conditions of time and temperature, an ampoule not submitted to the sterilization cycle, belonging to the same batch, as spore growth control (positive control).
- Examine ampoule colour and interpret results as per **EVALUATION TABLE**: a change of medium's colour from violet/clear to yellow/turbid indicates a microbial growth and therefore an unsuccessful sterilization. On the contrary, the persistence of the medium's initial colour (violet/clear) indicates absence of microbial growth and therefore a successful sterilization.

EVALUATION

Geobacillus stearothermophilus spores are killed off if the sterilization cycle has been efficient: in this case the ampoule's colour remains violet/clear even after incubation at 55-60 °C (131-140 °F) for 24-48 hours or for a different time validated by user.

If the sterilization cycle has not been efficient, spores partially survive and the ampoule's content turns yellow/turbid after incubation at 55-60 °C (131-140 °F) for 24-48 hours or for a different time validated by user.

The ampoule not submitted to the sterilization cycle and used as spore growth control has to turn yellow/turbid after incubation. On the contrary, the test must be repeated after having investigated the causes of the negative result.

EVALUATION TABLE

MEDIUM'S COLOUR	SPORE	STERILIZATION
Clear/violet	Killed off	Successful
Yellow / Turbid	Vital	Unsuccessful

AMPOULE TREATMENT AFTER USE

After use, sterilize the positive ampoules (yellow/turbid) in autoclave at 121 °C for at least 30 minutes and eliminate them in accordance with the procedures of the laboratory.

STORAGE

Store the product at 2-8 °C: in these conditions it maintains its validity until the expiry date indicated on the label.

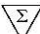
BIBLIOGRAPHY

- United States Pharmacopoeia latest edition.
- Deutsches Arzneibuch latest edition.
- European Pharmacopoeia latest edition.
- ISO 11138 and EN 866 latest edition.

PRESENTATION

PRODUCT	CODE	PACKAGING	Spores: CFU / ampoule	D _{121°C}
STERIL CONTROL GST E6 ampoules	91010	10 Ampoules	1-5x10 ⁶	1,5-3,0 minutes
STERIL CONTROL GST E6 ampoules	91050	50 Ampoules	1-5x10 ⁶	1,5-3,0 minutes
STERIL CONTROL GST E5 ampoules	91051	10 Ampoules	1-5x10 ⁵	1,5-3,0 minutes
STERIL CONTROL GST E5 ampoules	91052	50 Ampoules	1-5x10 ⁵	1,5-3,0 minutes
STERIL CONTROL GST E4 ampoules	91053	10 Ampoules	1-5x10 ⁴	1,5-3,0 minutes
STERIL CONTROL GST E4 ampoules	91054	50 Ampoules	1-5x10 ⁴	1,5-3,0 minutes

TABLE OF SYMBOLS

 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Caution, consult accompanying documents
 Use by	 Batch code	 Do not reuse

LIOFILCHEM Bacteriology Products

Via Scozia Zona Ind.le - 64026 Roseto D.A. (TE) - Italy

Tel. +390858930745

Fax +390858930330

Website: www.liofilchem.net

E-Mail: liofilchem@liofilchem.net

F12211
Rev.0 / 17.01.2005